

RFP-100A

RF Puncture Generator




Controlled Tissue Puncture
using **RADIOFREQUENCY ENERGY**

RFP-100A RF Puncture Generator

System designed to puncture tissue via radiofrequency (RF) energy

RF PUNCTURE (RFP) GENERATOR vs. RF ABLATION (RFA) GENERATOR

High impedance conditions are key to create a precise puncture in tissue, with minimal surrounding damage. The RFP Generator is designed to function at high impedance, whereas a typical RFA Generator is not.

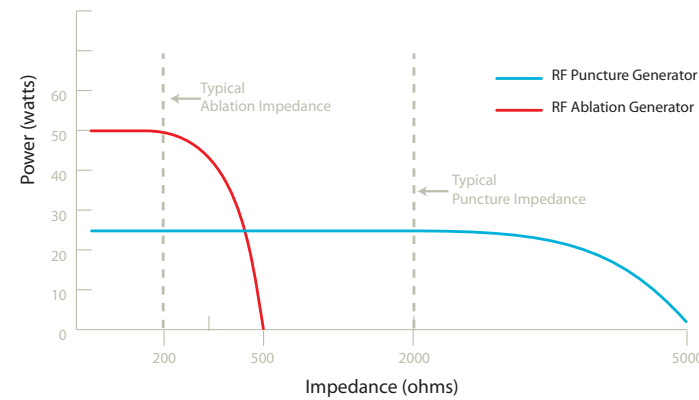


RF Puncture

- Objective: To create a small opening in tissue
- Occurs under the following conditions:
 - Low power (5-25 watts)
 - Short duration (1-3 seconds)
 - High voltage (270-400V)
- Impedance range: 2000-6000 Ω
- Minimal collateral damage to surrounding tissue


RF Ablation

- Objective: To create a lesion to destroy electrically conductive tissue
- Occurs under the following conditions:
 - High power (35-50 watts)
 - Long duration (60-90 seconds)
 - Low voltage (35-50V)
- Impedance range: 150-300 Ω
- Thermal destruction of surrounding tissue


High Impedance
High Voltage
Short RF Delivery time
Low Power

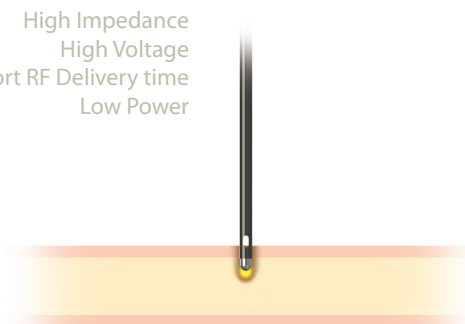

Low Impedance
Low Voltage
Longer RF Delivery time
High Power

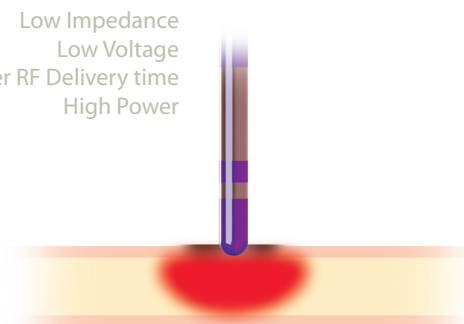


FEATURES

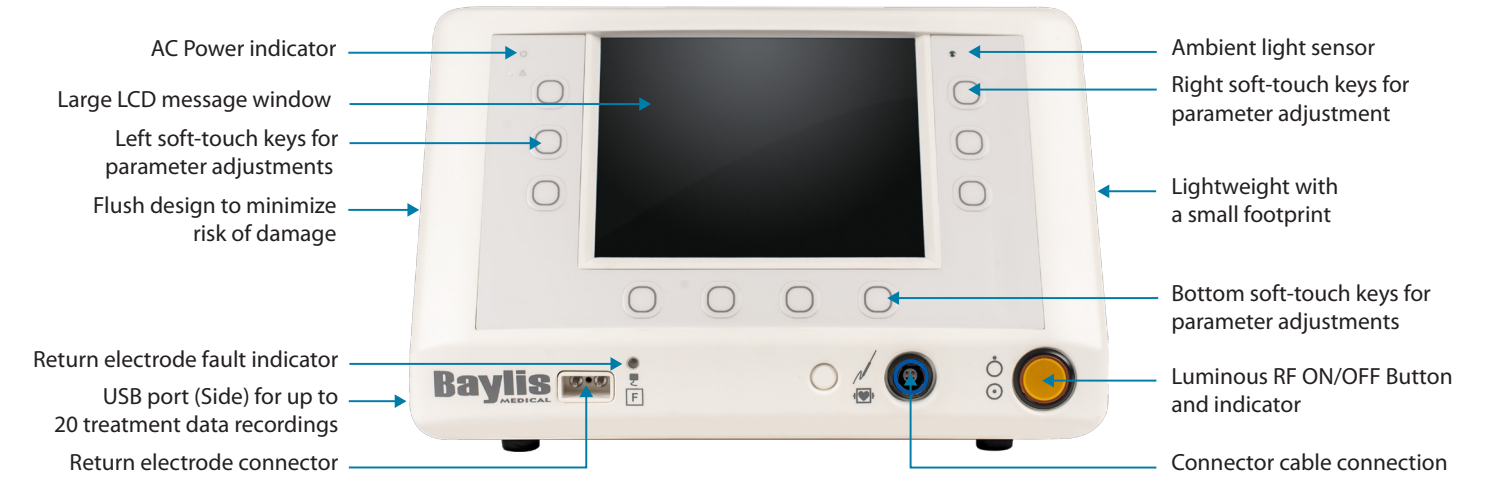




PRECISE: CUT SETTINGS OPTIONS

Improved cutting ability enables shorter RF activation time.

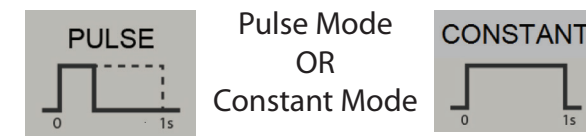

Setting options are tailored to user preference.

INTELLIGENT : USER INTERFACE

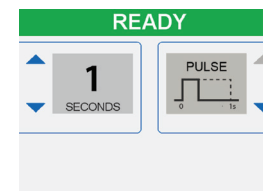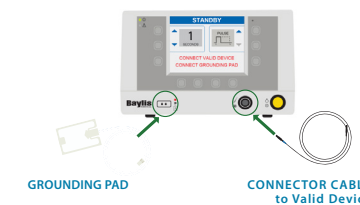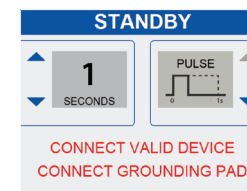

In Standby state, the user sets Time and Cut settings, and is asked to connect a valid device and grounding pad.

Generator automatically recognizes devices and makes available only appropriate modes.

The Ready Screen appears once the grounding pad and valid device are connected.

RFP-100A RF Puncture Generator

PUNCTURE WITH RF ENERGY

Specifications

Model number	RFP-100A
RF Energy	468 kHz, Sinusoidal Maximum output power of 50 Watts
Duty Cycle	Durations from 300 or 1000 ms \pm 5 ms
Count-up Timer	Settable from 1-10 seconds (Device dependent) Display resolution: 1 second
Dimensions	Width: 11.25 inches (28.5 cm) Length: 15.6 inches (39.6 cm) Height: 7 inches (17.8 cm)
Weight	20 lb. (9.1 kg)
Input Voltage	100-240 V~
Current Rating	5.0A, 50-60 Hz
Power Cord Length	10 feet
<i>WARNING: The RFP-100A RF Puncture Generator is designed and intended for use with devices designed by Baylis Medical Company</i>	

Accessories



RFP-100A Footswitch



RFP-Cart


Multi-platform design for maximal hospital value

NRG® Transseptal Needle




The NRG® Transseptal Needle is uniquely designed to assist the physician in gaining access to the left atrium.

PowerWire™ RF Guidewire

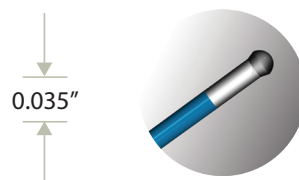

The PowerWire™ RF Guidewire is used to cross lesions in occluded blood vessels that are difficult to cross with a standard guidewire.*

Nykanen RF Wire

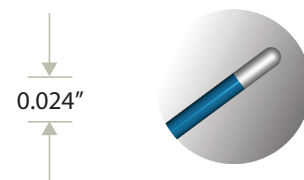

The Nykanen RF Wire is designed to create a controlled puncture in tissue.

*The PowerWire™ RF Guidewire is cleared by FDA to create a channel in totally occluded peripheral vessels 3 mm or greater.



© Copyright Baylis Medical Company Inc., 2014. Baylis Medical Company Inc. reserves the right to change specifications or to incorporate design changes without notice and without incurring any obligation relating to equipment previously manufactured or delivered. NRG, PowerWire and the Baylis Medical logo are trademarks or registered trademarks of Baylis Medical Company Inc. in the United States of America and/or other countries.

Before use, consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

CAUTION: Federal Law (USA) restricts the use of this device to or by the order of a physician.

Patents Pending and/or issued
CAR1054 01/15

Baylis

MEDICAL

Baylis Medical Company Inc.
5959 Trans-Canada Highway
Montreal, QC Canada H4T 1A1

Tel. : (514) 488-9801 / Fax : (514) 488-7209
www.baylismedical.com / info@baylismedical.com